Prior to use, carefully read the instructions for use.

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NL

## GC MI Paste Plus

TOPICAL CREAM WITH CALCIUM, PHOSPHATE AND FLUORIDE

For use by or under the supervision of a dental professional in the recommended indications.

**DESCRIPTION**  
GC MI Paste Plus is a water based cream containing RECALDENT™ with incorporated fluoride (CPP-ACP): Casein Phosphopeptide-Amorphous Calcium Phosphate Fluoride). The level of fluoride is 0.2% w/w (900ppm), which approximates that in adult strength toothpastes. When CPP-ACP is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localising bio-available calcium, phosphate and fluoride.

**IMPORTANT :**  
Saliva will enhance the effectiveness of CPP-ACP and the flavour will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

### A topical paste containing bio-available calcium, phosphate and fluoride

- Provides extra protection for teeth.
- Helps neutralize acid challenges from acidogenic bacteria in plaque.
- Helps neutralize acid challenges from other internal and external acid sources.

### CONTRAINDICATIONS

Casein phosphopeptides are derived from milk casein.  
Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity/allergy to benzene preservatives.

**DIRECTIONS FOR USE INDICATIONS**  
a. Following In-office bleaching procedures.  
b. After ultrasonic, hand scaling or root planing.  
c. Following professional tooth cleaning (P.T.C.).  
d. Hypersensitivity prevention and control.  
e. As an alternative means of applying fluoride topically in children aged 6 and above.  
f. During orthodontic treatment.  
g. For high risk caries patients.  
h. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögren's syndrome.  
i. For special needs adult patients.

### In-office application:

- Custom tray application:
  - Before use rinse the custom tray thoroughly under running water.
  - Extrude a generous layer of GC MI Paste Plus into the tray and apply to the upper and / or lower teeth.
  - Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
  - Remove the tray.
  - Instruction to the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Instruct the patient to retain for as long as possible (another 1 - 2 minutes) avoiding expectoration and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
  - Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
  - Any residual GC MI Paste Plus in the tray should be rinsed or brushed off under running water immediately after use.

### II. Non tray application:

- If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with compressed air.
- Apply GC MI Paste Plus to the tooth surfaces using an application swab, gloved finger or in difficult interproximal areas using an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
- Leave GC MI Paste Plus undisturbed for a minimum of 3 minutes.
- Then instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1-2 minutes) avoiding expectoration and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
- Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

### At home application:

#### III. Day time application after tooth brushing as recommended by a dental professional

- Apply a sufficient amount of GC MI Paste Plus to the upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
- Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
- Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
- Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

### IV. Night time application after tooth brushing as recommended by a dental professional

- Note:  
The overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12.
- Apply a sufficient amount of GC MI Paste Plus to the upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
  - Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
  - Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
  - Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

### FLAVOURS

Strawberry (S), Melon (M), Vanilla (V), Mint (I), Tutti-Frutti (T)

### STORAGE

Store in a cool dry low humidity area away from direct sunlight (8-25°C) (46-47.7°F). (Shelf life: 2 years from date of manufacture)

### PACKAGES

Tube containing 40g (35ml)  
1. Assorted Package  
10 x 40g (2 each of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti)

2. Single Flavour Package  
10 x 40g in each flavor  
Vanilla, Strawberry\*\*, Melon\*\*, Tutti-Frutti\*\*, Mint\*\*

\*\*available in Europe only

3. Refill Package

1 x 40g tube in each flavour

NOTE

GC MI Paste Plus with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia.

\*RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

### CAUTION

- Always replace the cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
- It is not recommended to use GC MI Paste Plus on patients under the age of 6.
- The overnight at home application is not recommended for children under 12.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- In case of spillage on clothes, wash off with water.
- If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzene preservatives, or to some other component of the product such as a flavouring agent. In this event, discontinue the use of the product and refer to a physician.

Last revised : 07/2008

'GC.'

MANUFACTURED BY  
GC CORPORATION  
7-1 Heiwa-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY  
GC CORPORATION  
7-1 Heiwa-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.  
Research-Park, Interleuvenlaan 13, B-3001 Leuven, Belgium  
TEL +32 3 265 80 80

GC AMERICA INC.  
3720 Peachtree Industrial Blvd., Alpharetta, GA 30090 U.S.A.  
TEL +1 770 597-0900

GC ASIA DENTAL PTE. LTD.  
18 Loyang Way #06-27 Singapore 598724  
TEL +65 6546 7588

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#### III. Non tray application:

1. If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with compressed air.
2. Apply GC MI Paste Plus to the tooth surfaces using an application swab, gloved finger or in difficult interproximal areas using an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
3. Leave GC MI Paste Plus ungestört für einen Minimum von 3 Minuten.
4. Then instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1-2 minutes) avoiding expectoration and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
5. Ask the patient to expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

#### At home application:

1. Day time application after tooth brushing as recommended by a dental professional:
  1. Apply a sufficient amount of GC MI Paste Plus to the upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
  2. Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
  3. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
  4. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

#### IV. Night time application after tooth brushing as recommended by a dental professional

- Note:  
The overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12.
1. Apply a sufficient amount of GC MI Paste Plus to the upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
  2. Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
  3. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
  4. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

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**DESCRIPTION**  
GC MI Paste Plus is a water based cream containing RECALDENT™\* with incorporated fluoride (CPP-ACP): Casein Phosphopeptide-Amorphous Calcium Phosphate Fluoride). The level of fluoride is 0.2% w/w (900ppm), which corresponds to that in adult strength toothpastes. When CPP-ACP is applied in the oral environment, it will bind to plaque, plaque, bacteria, hydroxyapatite and soft tissue, isolating bio-available calcium, phosphate and fluoride.

**IMPORTANT :**  
Sina will enhance the effectiveness of CPP-ACP and the flavour will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

A topical paste containing bio-available calcium, phosphate and fluoride

- Provides extra protection for teeth
- Helps neutralize acid challenges from acidicogenic bacteria in plaque
- Helps neutralize acid challenges from other internal and external acid sources

**CONTRAINDICATIONS**

Casein phosphopeptides are derived from milk casein.

Do not use this material on patients with a proven or suspected milk protein allergy and/or a sensitivity or allergy to benzoteze preservatives.

**DIRECTIONS FOR USE****INDICATIONS**

a. Following in-office bleaching procedures

b. After orthodontic treatment or root planning.

c. Following professional scaling and cleaning (PTC.)

d. Hypersensitivity prevention and control.

e. As an alternative means of applying fluoride topically in children aged 6 and above.

f. During orthodontic treatment.

g. For high risk caries patients.

h. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögren's syndrome.

i. For special needs adult patients.

**In-office application:**

1. Before placing the custom tray thoroughly under running water.

2. Apply a generous layer of GC MI Paste Plus into the tray and apply to the upper and / or lower teeth.

3. Leave the tray undisturbed in the mouth for a minimum of 3 minutes.

4. Remove the tray.

5. Instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Instruct the patient to retain for as long as possible (another 1-2 minutes) avoiding expectation and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.

6. Advise the patient to expectorate thoroughly and if possible avoid rinsing.

7. Advise GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

8. Any residual GC MI Paste Plus in the tray should be rinsed or brushed off under running water immediately after use.

**II. Non tray application:**

1. If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with compressed air.

2. Apply a generous layer of GC MI Paste Plus onto the tooth surface using an application swab, or in difficult interproximal areas using an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.

3. Leave GC MI Paste Plus on the tooth surface using an application swab, or finger or in difficult interproximal areas using an Interproximal Tooth Cleaning Brush.

4. Instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1-2 minutes) avoiding expectation and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the result.

5. Advise the patient to expectorate thoroughly and if possible avoid rinsing.

Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

**At-home application:****III. Day time application after tooth brushing as recommended by a dental professional**

1. Apply a sufficient amount of GC MI Paste Plus to upper and lower teeth.

2. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip.

For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus for a minimum of 3 minutes.

2. Leave GC MI Paste Plus on the tooth for a minimum of 3 minutes.

3. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectation (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the result.

4. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

**IV. Night time application after tooth brushing as recommended by a dental professional**

Note: The overnight use is especially recommended for high-risk adult patients, but not recommended for children under the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip.

For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus for a minimum of 3 minutes.

2. Leave GC MI Paste Plus on the tooth for a minimum of 3 minutes.

3. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectation (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the result.

4. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

**FLAVOURS**

Strawberry (S), Melon (M), Vanilla (V), Mint (I), Tutti-Frutti (T)

**STORAGE**

Store in a cool dry low humidity area away from direct sunlight (8-25°C) (46-77°F).

(Shelf life: 2 years from date of manufacture)

**PACKAGES**

Tube containing 40g (35ml)

1. Assorted Package

10 x 40g (2 each of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti)

2. Single flavor package

10 x 40g in each flavor

Vanilla, Strawberry, Melon, Tutti-Frutti, Mint

3. Refill Package

1 x 40g tube in each flavor

**NOTE**

GC MI Paste Plus with its unique CPP-ACP formulation was developed at

The School of Dental Science, The University of Melbourne, Victoria,

Australia. "RECALDENT" is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

**CAUTION**

1. Always replace the cap after use and remove any residual paste or moisture from the tube neck or inside the cap.

2. It is not recommended to use GC MI Paste Plus on patients under the age of 6.

3. The overnight at home application is not recommended for children under 12.

4. In case of contact with eyes, flush immediately with water and seek medical attention.

5. Do not swallow.

6. If any angiodema symptoms are experienced, this may indicate sensitivity or allergy to the benzoteze preservatives, or to some other component of the product such as a flavoring agent. In this event, discontinue the use of the product and refer to a physician.

Last revised : 07/2008

**GC**  
MANUFACTURED BY  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED BY  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU GC EUROPE N.V.  
Respoortlaan 13, B-3001 Leuven, Belgium

TEL : +32. 16. 39. 80. 50

PRINTED IN JAPAN

CE

MANUFACTURED BY  
GC CORPORATION

76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

PRINTED IN JAPAN